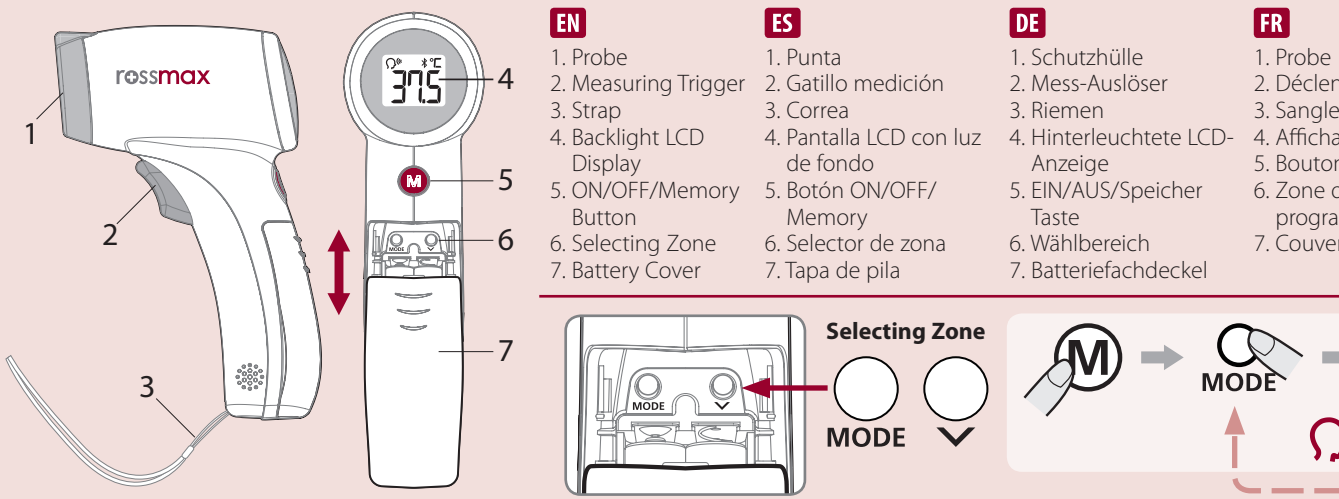
EN English

Introduction

Rossmax non-contact temple thermometer uses an infrared detector to detect body temperature from the temple for adult and child in the hospital, home or public place. This thermometer could also detect object temperature.

The quality of the infrared temple thermometer has been verified and conforms to the provisions of the EC Council directive 93/42/EEC (Medical Device Directive) Annex I essential requirements and applied harmonized standards. Comply with ASTM E1965-98(2016). This thermometer converts the temple temperature to display its 'oral equivalent' (according to the result of the clinical evaluation).

Please read the instruction manual carefully before using the product and be sure to keep this manual.

Function	The thermometer has been designed for practical use. It's not meant to replace a visit to the doctor. Please also remember to compare the measurement result to your regular body temperature. The object mode shows the actual, unadjusted surface temperatures, which is different from the body temperature. It can help you to measure any object other than body surface, for example the baby's milk. If thermometer detects a body temperature >37.5°C (99.5°F) there will be four beeps sound with flashing redlight to warn the user for potential fever.
Temple Temperature	
Object Mode	
Fever Alarm	

Setting your thermometer

- Open the battery cover.
- Press "ON/OFF/Memory" button to power on the device.
- Enter the setting mode by pressing the "MODE" button to switch Temple/Object temperature, Celsius/Fahrenheit scale, and adjust Bluetooth and beep sound in order.
- Change the measuring target between the temple and object by pressing the "MODE" button to set a new measuring target, the measuring site icon on the temple will blink twice, then the "MODE" button to choose whether automatic Bluetooth data transfer and beep sound is activated (Bluetooth symbol ⌘ on or ⌘ off) or deactivated (Bluetooth symbol ⌘ OFF b/ OFF).
- Your thermometer is ready for measurement after all symbols appear or pressing ON/OFF/Memory button again.

Note:
The initial setting is Temple temperature mode, Celsius scale, Bluetooth and beep sound activated as default.
1. Always make sure the probe is clean and without damage.
2. Press "ON/OFF/Memory" button to turn on the device.
3. Make sure the measuring site icon ⌘ is shown on the display before taking measurement.
4. Vertically aim the thermometer at the temple with a distance of less than 10 cm. If the temple is covered with hair, sweat or dirt, remove the obstacle from the temple to improve the measuring accuracy.
5. Press the "Measuring Trigger" button to take temperature. Measurement is complete when you hear a long beep sound.
6. Read the recorded temperature from the LCD display.
7. If Bluetooth has been activated, the data is automatically transferred to the App after successful completion of the binding process, please see Data Transfer via Bluetooth.

Specification	Temple mode: 34~42.2°C (93.2~108°F) Object mode: 34~42.2°C (93.2~108°F) For temple mode: ±0.3°C (0.5°F) during 34~42.2°C (93.2~108°F). For object mode: ±0.3°C (0.5°F) when measuring the object temperature between 6~99.9°C (41~32~199.9°F).
Accuracy	Temple mode: 16~40°C (60.8~104°F), RH <85%; Object mode: 16~40°C (60.8~104°F), RH <85%; Temperature: -25~55°C (-13~131°F), RH <85%; 700~1060 hPa
Operating condition	Storage and transportation condition: 700~1060 hPa Memory: 9 sets Display resolution: 0.1 Weight: 160g (without battery) Size: 10.4cm(L) x 5.2cm(W) x 1.5cm(H) Auto shutdown: 60 sec Safety classification: ⌘ Type BF equipment IP classification: ⌘ Protection against water and dust
Disposal of device and batteries according to local regulations	

How to measure temple temperature

- Always make sure the probe is clean and without damage.
- Press "ON/OFF/Memory" button to turn on the device.
- Make sure the measuring site icon ⌘ is shown on the display before taking measurement.
- Vertically aim the thermometer at the temple with a distance of less than 10 cm. If the temple is covered with hair, sweat or dirt, remove the obstacle from the temple to improve the measuring accuracy.
- Press the "Measuring Trigger" button to take temperature. Measurement is complete when you hear a long beep sound.
- Read the recorded temperature from the LCD display.
- If Bluetooth has been activated, the data is automatically transferred to the App after successful completion of the binding process, please see Data Transfer via Bluetooth.

Note:
1. Applications include temperature measurements for Water, Milk, Cloth, Skin or other object.
2. This mode shows the actual, unadjusted surface temperatures, which is different from the body temperature.
3. The object mode shows the actual, unadjusted surface temperatures, which is different from the body temperature.
4. Vertically aim the thermometer at the temple with a distance of less than 10 cm. If the temple is covered with hair, sweat or dirt, remove the obstacle from the temple to improve the measuring accuracy.

Data Transfer via Bluetooth

Pairing the device with your Smartphone

- To begin using Bluetooth for the first time, please visit the website at <http://www.rossmax.com> for the initial set-up instructions.
- Download and install the applicable APP onto your smartphone.
2. bind this device with your smartphone, turn on the device, Bluetooth and the App of smartphone, and follow set-up and binding instructions.
3. The Bluetooth symbol ⌘ will flash on the display that indicates the binding is proceeding.
 - If the binding is successful, the Bluetooth symbol ⌘ will illuminate on the display and the current measured temperature will automatically be transferred to the App.
 - If the binding has failed, the Bluetooth symbol ⌘ will keep flashing until the device shuts off automatically after 1 minute, and the current measured temperature will not automatically be transferred to the App. Please rebind this device with your smartphone.
- Notes:
 - 1. Unbinding your device will not delete the data in the App.
 - 2. If you rebind your smartphone with the device, all prior reading history stored on the phone App will be retained.
 - 3. Bluetooth data transfer will reduce the battery capacity.
 - 4. Electromagnetic interference: The device contains sensitive electronic components. Avoid strong electrical or electromagnetic fields in the direct vicinity of the device (e.g. mobile telephones, microwave ovens). These may lead to temporary impairment of measurement accuracy.

Memory Function
There are total 9 set memories for measurement records. If the reading of the thermometer is within the normal temperature range of 34°C to 42.2°C (93.2°F to 108°F), when the measurement is done, the measurement data is saved into memory.
- Press "ON/OFF/Memory" button to repeatedly recall each temperature memory.

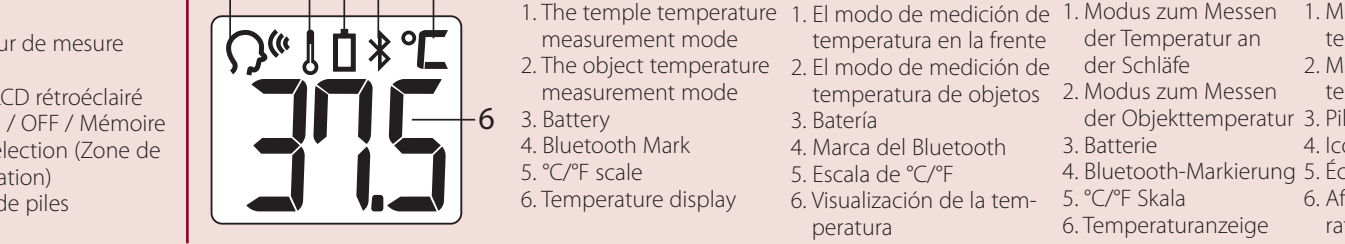
Care and cleaning

- Storage temperature range: It should be stored at room temperature between -25~+55°C (-13°F~131°F), RH <85%, 700~1060hPa.
- Keep the unit dry and away from humidity and direct sunlight.
- The probe should not be submerged into liquids.
- Please check the device if it is damaged or if you can't make sure of its safety. Please send the complete device to the nearest retailer for re-calibration.
- Holding the thermometer too long may cause a higher ambient temperature reading of the probe. This may make the body temperature measurement lower than usual. Please try to not touch the thermometer probe part for avoiding the incorrect reading.

Installing Batteries

- Open the battery cover: use thumb to push battery cover out.
- Install or replace 2 AAA Sized batteries in the battery compartment according to the indications inside the compartment.

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